



ANESURETM-S Sidestream Multigas Analyzer

Features

- A. Employs sidestream method to provide CO2 waveforms, EtCO2, Respiratory rate, as well as the waveforms, inspiration and expiration concentration values of nitrous oxide and one of the five anesthetic gas agents (Halothane, Enflurane, Isoflurane, Sevoflurane or Desflurane) selected manually by the user.
- B. PLUG AND PLAY. The host only needs a standard Remo socket and Internal serial port without interior space and circuit.
- C. HIGH ACCURACY. Meticulously chosen narrow band optical filter, infrared source and delicately designed hardware electrical circuitand software algorithm ensure to realize high SNR (signal to noise ratio) gas concentration measurement and ensure the measurement accuracy in the range of -5-50°C of temperature and within 0-20% of concentration.
- D. AUTOMATIC PRESSURE COMPENSATION. Within very wide pressure range of 400-860mmHg, it maintains measurement accuracy by onboard barometric sensor
- E. LONG LIFE PUMP REDUCES PERIODIC MAINTENANCE
- F. IMMEDIATELY TO ACTIVATE. Power on in 4 seconds, to get accurate data.
- G. ADULT/PAEDIATRIC,OR NEONATE AIRWAY ADAPTER can continuously work in long time and be used repeatedly, reducing costumer's usage cost.
- H. Low cost Oxygen sensor can be cooperative used and oxygen concentration measure can be realized easily.
- I. COMPREHENSIVE OEM RESOLUTION OF CO2 AND MUTIGAS. Multiple communication protocols. Moreover, our long-term and accumulated technology and experience easily help you to realize upgrading technology and performance.

Performance of the module

Technique: Non-dispersive infrared gas analysis, Multi-channel

infrared detector, Infrared resource, no moving parts

Storage: -40 to 70°C, <90% RH, non-condensing Operation: -5 to 50°C, 10 to 90% RH, non-condensing

Circumstance pressure: 55-115kP

Power demand: 5 V±5% (max ripple 200 mVp-p)

Power consumption: <180mA

Interface: RS-232/ Standard E-8-1,38.4 kBaud,10ms

data interval(KM) or customized communication protocol.

CO2 measurement range:0-19.7%,150mmHg,or 0-20kPa measurement accuracy:

0 - 40 mmHg ± 2 mmHg

41 - 70 mmHg ± 5% of reading

71 - 100 mmHg ± 8% of reading

101 - 150 mmHg ± 10% of reading

N20 measurement range:0-100%

measurement accuracy: ±(2 vol% + 2% of reading)

HAL, ENF, ISO measurement range: 0-12%

measurement accuracy:

0-8%: $\pm (0.2 \text{ vol}\% + 5\% \text{ of reading})$

8-12%: unspecified

SEV measurement range: 0-15%

measurement accuracy:

0-10%: $\pm(0.2 \text{ vol}\% + 5\% \text{ of reading})$

10-15%: unspecified

DES measurement range:0-25%

measurement accuracy:

0-18%: $\pm(0.2 \text{ vol\%} + 5\% \text{ of reading})$

18-25%: unspecified

Respiration rate measurement range: 2-150 bpm measurement accuracy: 1% ±1bpm

Water filter Detection: Insertion automatically turns sampling pump on.

Removal automatically turns sampling pump off

Flow set: 50-150ml/min selectable

Exhaust Port: Yes

O2 Compensation: HOST supplied

Barometric Pressure compensation: 400 to 860mmHg automatic
Warm-up time: To reach designed accuracy 97% within 8s, to reach

designed accuracy within 20s

Stability:

Shortly drifting less than 1mmHg, after 4 hours' power on. Longly drifting does not influence nominal accuracy after

continuous 140 hours' power on.

Calibration: No routine user calibration required

Zero: Very few zero only when needed

EtCO2 calcualtion: every breath, 10 or 20s averagely

Water Resistance: IPX4-Splash-proof(when water filter inserted)
Shock Impact: IEC TR 60721-4-7 Class 7M3, EN60068-2-27

Shock, EN60068-2-64 Random vibration

Regulatory Designed to meet the following standards:

IEC 60601-1-2, EN55011 - CISPIR 11 Class B

IEC 61000-4-2 Electrostatic Discharge Immunity,

IEC 61000-4-3 Radiated Immunity,

Designed to comply with 93/42/EEC (MDD CE Marking), FDA Standards, Minimum Performance and Safety Requirements for Capnom-

eters ISO80601-2-55

Configurations of	f the accessories		Luer Connector	Filter membance	transparent	Storage	Term	Patient type	time	packaging
Water Filter		T3 Filter 664511	Female			0.5ml	Long	Adult Pediatric Infant	8 hours	10
		T4F Filter 664522	Female			1.3ml	Long	Adult Pediatric Infant	24hours	25
		T4M Filter 664523	Male			1.3ml	Long	Adult Pediatric infant	24hours	25
			Luer Connector	Length	Patient	O2 tubing	Adult	Pediatric	Infant	Unit packaging
Sample line		Sample line 662503	Male	2.4m	Intubated					10
	C	O2 Nasal tube 660423	Male	2.4m	Unintubated					10
	C	O2 Nasal tube 660424	Male	1.5m	Unintubated					10
	CO2/	O2 Nasal tube 660425	Male	2.0m	Unintubated					5
	CO2	O2 Nasal tube 660426	Male	2.0m	Unintubated					5

Airway Adapter



L Airway Adapter 662532 10pcs/Box



Kingst Commercial and Trade Co. Ltd

Add: The 6th Floor,No.2-7,Road Beibinhe in Xibianmen,Dist.Xicheng,Beijing,P.R.C.

Code: 100045

Tel: 0086 10 68156857/68158013 Email: mail@ekingst.com